


Dentemp® FAQ

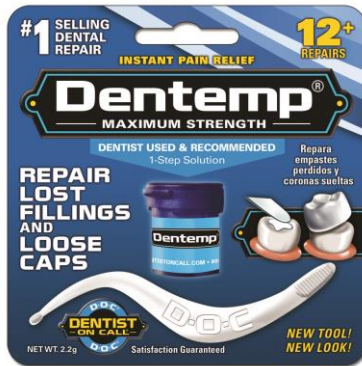

Q – Should I use Dentemp® if I have throbbing pain or swelling in the affected area?

A – No. You should consult your dentist immediately. This may be an indication of infection.

Q – When replacing a cap or crown, should I put the cement directly into the center of the cap or crown?

A – No. Place a small amount of the cement on the upper inside wall of the cap or crown. Place the cap or crown on the tooth and press firmly for several minutes.

Q – When replacing a lost filling, how do I get the ball of cement to stay in the hole?

A – Place the ball on the tip of your finger and put the ball into the hole of your tooth. Then disengage the cement from your finger by scraping your finger along the side of your tooth.

Q – Should Dentemp® be used on a post or implant?

A – It differs from case to case. Before using on a post or implant, seek your dentist's advice.

Q – Can Dentemp® be used to repair a broken tooth?

A – No. You should consult your dentist.

Q – Can Dentemp® be used to repair a denture?

A – No. You should use [Repair-It](#) or consult your dentist.

Q – Will I get sick if I accidentally swallow Dentemp®?

A – No.

Q – Is there any type of crown on which Dentemp® should not be used?

A – No. Dentemp® can be used on any type of crown.

Q – What is Eugenol?

A – Eugenol is the active ingredient in clove oil. Eugenol helps to alleviate pain.

Q – Can I use Dentemp® if I am nursing? If I have a latex allergy? If I have high blood pressure?

A – Yes. Yes. Yes.

Q – Can I use Dentemp® while pregnant?

A – Consult your obstetrician.

Q – Is Dentemp® gluten free?

A – Yes.

Q – Can Dentemp® be used on children?

A – Consult your dentist.