

REFILIT[®]

Repairs Lost Fillings

Questions? / ¿Preguntas? **800-238-0220**
visit us at:
www.dentemp.com

Refilit[®] - the Best Dental Repair Material Available! **DENTIST RECOMMENDED • NO MIXING • NO WASTE**

REFILIT[®] provides fast relief of pain and discomfort until you can see your dentist. Clinically tested, **REFILIT[®]** has a pleasant cherry taste, and is so safe and effective *"you can eat on it"*.

⚠ READ DIRECTIONS CAREFULLY BEFORE USING KEEP OUT OF REACH OF CHILDREN

Caution: This product should not be used if throbbing pain or swelling in affected area exists. Consult your dentist immediately. This product is sold as a **temporary remedy**. See your dentist as soon as possible for permanent work to be performed. Failure to do so may cause infections and/or serious injury.

To Replace a Lost filling: Wash hands thoroughly. Rinse out cavity with warm water. Do not dry; **leave moist**. Use applicator to remove a small amount of material from vial, form into a ball with your fingers and press firmly into cavity. Bite down several times to insure proper bite. **REMOVE ANY EXCESS MATERIAL. RINSE OUT MOUTH WELL WITH WATER.** Allow 1 hour to set before eating.

Store at room temperature with container closed to keep material soft.

Any leftover material may be used as long as the material is soft and pliable. Once the material hardens it should be discarded. Lot code on container.

Ingredients: Calcium Sulfate, Barium Sulfate, Dimethoxytetraethylene Glycol, Zinc Oxide, Ethylmethacrylate Polymer, Cherry Flavour, Acesulfame-K.



© 2012 **Majestic Drug Co., Inc.**
South Fallsburg, NY 12779 USA

MPF 120612

